


# VIVUS CF4 User Manual

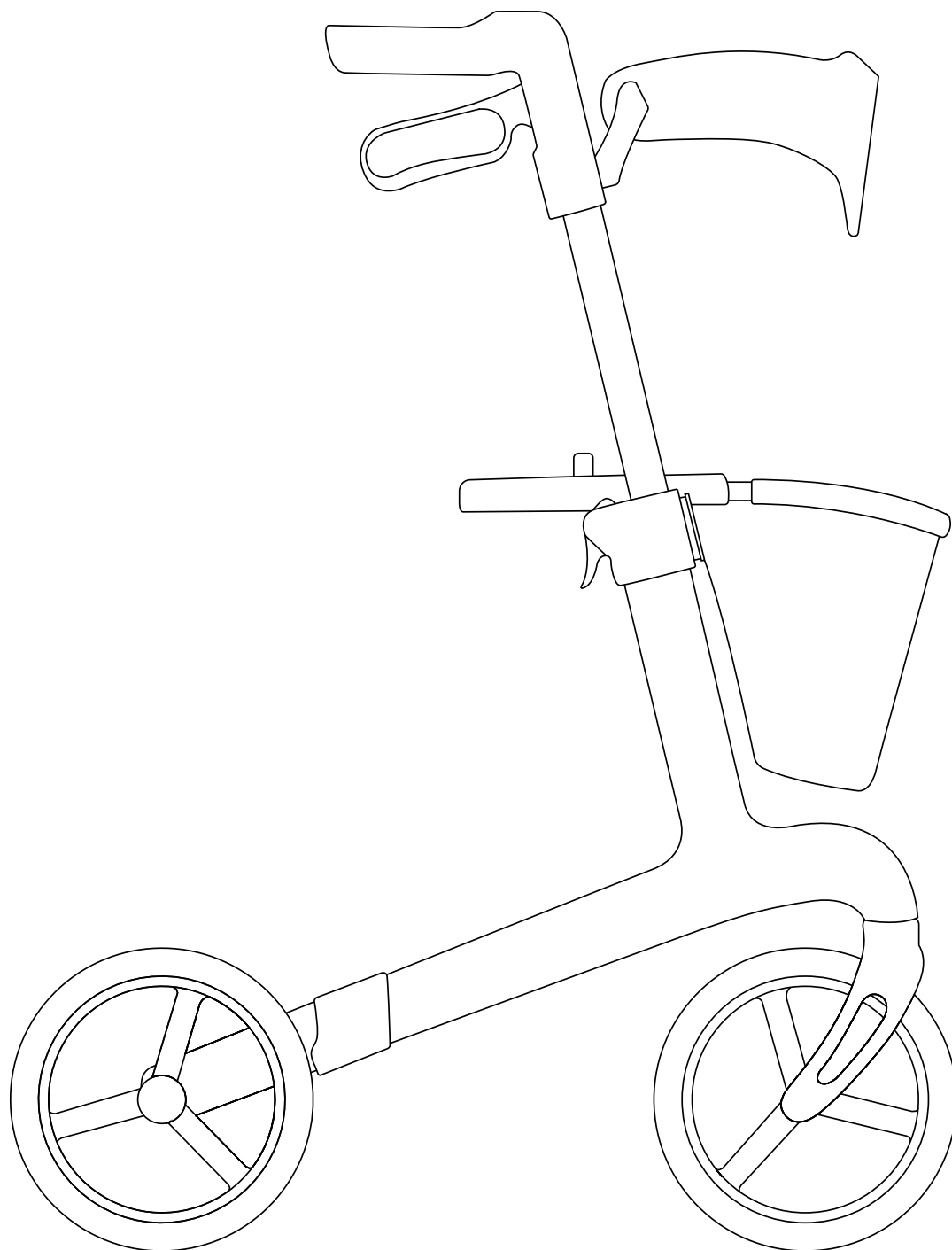

**Motion**

**CE UK  
CA**

## 1. Introduction

Thank you and congratulations on purchasing your new Motion Healthcare Vivus CF4 Rollator.


Your new device is designed to provide transportation indoors and outdoors for a person whose ability to walk is impaired, but who in terms of eyesight, dexterity and cognitive ability are able to safely operate the device.

We pride ourselves on providing safe, comfortable, and high-quality products and our goal is to ensure your complete satisfaction, we sincerely hope we achieve that, and you enjoy your Motion Healthcare device.

Please read and observe all warnings and instructions provided in this user manual before you operate the various functions of your device and please retain this user manual for future reference.

If you have any questions, please contact the company you purchased from.

## 2. Getting to know your rollator

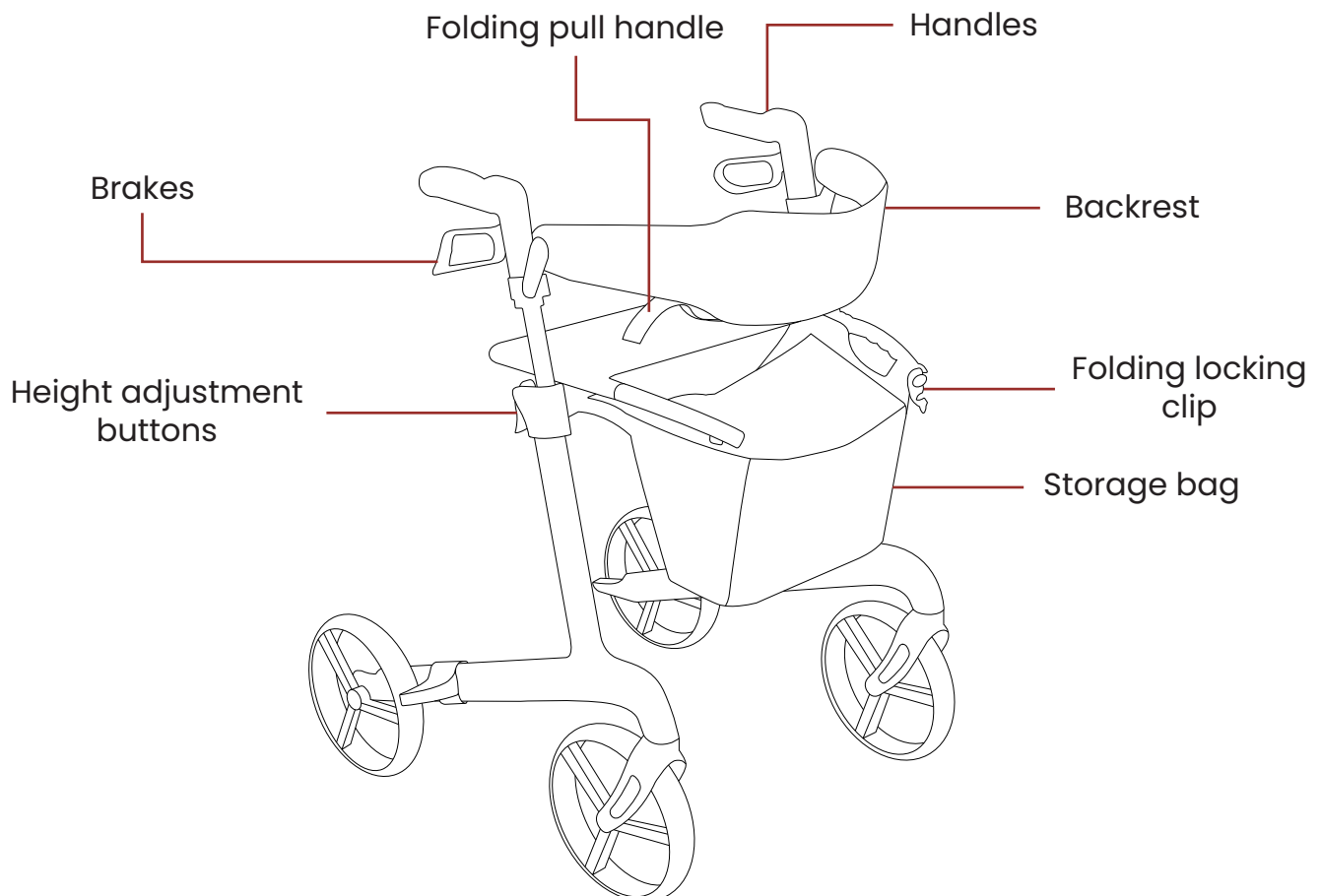

### **3. Important safety information**

Before you operate your device, please read and understand all of the information provided for your safety. As with operating any device, only propel yourself with full alertness to ensure safety. Please observe the warnings below to avoid any damage or injury resulting from improper use of your device. You are responsible for taking proper safety measures and we will not be held liable for personal injury and/or product damage resulting from improper use of the device.

- Please consult your doctor routinely to ensure there are no health or physical conditions that may limit or impair your ability to safely operate your device.
- Consult your doctor if you are taking prescription or over-the-counter medications to see if any side effects may limit or impair your ability to safely operate your device.
- Do not operate your device while you are under the influence of alcohol, as this may impair your ability to operate it safely.
- Do not use the device other than for its original purpose. Avoid any use, such as weight training, sports, athletics, hauling, moving or towing anything, that may lead to safety hazards and undue stress on the device.
- Your device is designed for one person only. Do not carry passengers.
- We strongly recommend that you do not smoke cigarettes using your device. You must adhere to the following safety guidelines if you decide to smoke cigarettes while using your device: do not leave lit cigarettes unattended. Keep ashtrays at a safe distance from the seat covers. Always make sure cigarettes are completely extinguished before disposal.
- Be aware that washing of any upholstered items may reduce the fabric's fire resistance.
- Replace worn or damaged upholstery immediately to reduce the risk of flammability.
- Do not modify your device. For any accessories, please contact an approved company for installation.
- The addition of accessories to your device may change certain specifications such as overall weight, size, and /or the centre of gravity. Please take note of how some of the changes may damage any objects around you.
- To prevent the device from rolling uncontrollably on its own, lock the breaks on any incline or decline.
- Keep yourself, clothing, and all other objects away from the wheels while moving. Do not allow any objects to drag behind the device when moving. Loose-fitting clothing or other objects can get caught in the wheels.

### **4. Mounting & Dismounting Safely**

To safely mount and dismount your device always make sure the brakes are locked on, if you fail to do so you may move unexpectedly. Move your device as close as you can to the seat you are transferring from. Transfer as far back onto the seat surface as you can, this will reduce the risk that you will miss the seat or fall.

The full correct transfer method varies based on your personal circumstances so work with your health care professional to learn how to position your body and how to support yourself during a transfer.

We also advise you have someone help you until you are sure you can safely transfer on your own.

## **5. Moving Safely**

- Always obey all of the local pedestrian and traffic rules. Wait until your path is clear of traffic, and then proceed with extreme caution. Use of safety equipment such as reflectors/reflective clothing, lights, and safety flags is advisory.
- Every device is different. Take the time to learn the feel of this device before operating.
- Get to know the areas where you plan to use device. Look for hazards and learn how to avoid them.
- To prevent tipping reduce your speed when turning and do not make any sharp turns. Always maintain a stable centre of gravity while turning and avoid shifting your weight in the opposite direction of a turn.
- Do not travel up or down potentially hazardous surfaces and/or inclines, including but not limited to areas covered with snow, ice, cut grass, or wet leaves.
- Avoid any change of position or movement that may change your centre of gravity while the device is moving.

When operating your device always be aware of the following:

### **Doorways**

Never use the sides of a doorway to pull yourself through. The safest way is to line up square to the door with both hands on your rims, ensure you have ample space each side and then proceed.

### **Leaning**

Do not lean over the side of your device. Always be careful when leaning backwards and ensure you are in full control if you ever tip your device backwards.

### **Inclines**

Never turn on an incline to avoid tipping. Avoid stopping on an incline. Do not exceed the maximum safe incline of the device.

### **Kerbs**

Whenever possible get help from an attendant when mounting kerbs. Avoid hard impacts when descending kerbs.

### **Stairs**

Always use two or more assistants. Tilt the device to its point of balance. One assistants (from behind) holds the device against the first step and safely lifts up. The second assistant (in front) lifts from a fixed part of the frame to above the stair height and holds it while the first assistant moves up one step. Repeat one step at a time until both assistants safely reach the summit of the stairs.

## **6. Safe Transportation**

When transporting your device always be aware of the following:

- Do not use your device as a seat in any motor vehicle.
- Always secure the device when it is being transported.
- Never transport the device in the front seat of a vehicle as it may move and distract the driver.
- Never sit on your device while it is in a moving vehicle.
- Do not tow your device with a car. Towing may exceed the maximum speed threshold, resulting in damage to critical components of the device.

## **7. Folding & Unfolding**

### **Folding**

1. Stand behind your device.
2. Lift up the red handle located on the middle of the seat.
3. Push the two handles together to the middle.
4. Lock the device together with the red clip located on the front of the device.

### **Unfolding**

1. Unlock on the red clip located on the front of the device.
2. Pull the two handles apart.
3. Push the sides of the seat until the device is locked into place.

## **8. Applying the brakes**

To apply the brakes, push down on the handle triggers until they click into place and lock the brakes.

## **9. Adjusting handle height**

To adjust the height of the handles:

1. Engage the device's brakes.
2. Pull and hold the adjustment button.
3. Adjust the height of the handles by raising and lowering.
4. Release the adjustment button.
5. Push down on the handles to check it is locked.
6. Adjust the other handle to the same height.
7. Check both handles are the same height.

## **10. Seatbelt Fitting & Adjustment**

This device is not fitted with a seatbelt.

## **11. Daily Checks**

Check the following items before using your device. If you find anything abnormal or that you are unsure of contact the company you purchased from for further advice or inspection before using your device.

### **Wheels**

- Ensure there are no cracks or deformities on the tyres of your device
- Ensure the wheels rotate without wobbling
- Ensure there are no foreign objects in the tyre treads
- Ensure there is no excessive or uneven wear to the tyre tread
- Ensure there are no unusual noises coming from your device

### **Seat**

- Ensure there is no unusual movement
- Ensure the upholstery is not excessively worn

### **Brakes**

- Ensure the brakes stop the wheelchair from moving when locked
- Ensure the brake levers move with no obstruction and apply the brakes when pulled

## **12. Cleaning, Maintenance & Disposal**

### **Cleaning**

To clean your device, use a soft brush to remove any dirt or dust and use a damp cloth and soft detergent to wipe all surfaces clean. Allow your device to dry naturally.

- Do not use a hose pipe, jet wash or splash water directly onto your device
- Do not use abrasive or scouring materials or liquids when cleaning

### **Maintenance**

Ensure the daily checks are made on your device. Apart from cleaning and daily checks it is recommended that all maintenance is completed by an approved company.

### **Disposal**

It is recommended that your device is disposed of through an approved company. For more information contact the company you purchased from.

### 13. Warranty

All Motion Healthcare products are sold to our partners with a warranty, they in turn will sell the product to you with a warranty, these are two separate agreements.

To find out specific details of the warranty you receive or to make any claim please contact the company you purchased from.



#### **Motion Healthcare Ltd**

Units 6 & 7 Concord Farm  
School Road  
Rayne, CM77 6SP



#### **Intco Medical (HK) Co., Ltd**

Flat/RM 19C, Lockhart Centre,  
301-307 Lockhart Road, Wan Chai,  
Hong Kong, China



#### **MedNet EC-REP GmbH**

Borkstrasse 10  
48163 Muenster  
Germany



#### **Medimap Ltd**

2 The Drift  
Suffolk  
Thurston  
IP31 3RT  
United Kingdom

## 14. Specifications

Total Weight	6.6 kg
Weight Limit	20.5 stone
Seat Width	41.5 cm
Seat Height	55 / 64 cm
Wheels	23 cm









**Motion**

[www.motionhealthcare.co.uk](http://www.motionhealthcare.co.uk)

0844 257 8150